Selbstklebende Elektroden Self-adhesive electrodes

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Self-adhesive electrodes

size: 40 x 40 mm, 2 mm plug | item no: 46501 | content: 4 pieces

Usage instructions

1. Connect the wire with the electrodes.
2. Take the electrodes off of the protection foil and attach them to clean, dry and unirritated skin. Press them on tightly, to make sure the electrodes have continuous skin contact.
3. Now link the wire to your electrostimulator.

Maintenance and storage

Please lift up the electrode at one corner and do NOT pull at the wire to remove the electrode from your skin. Now re-attach the electrodes to the protection foil and keep them in the re-usable package at a cool place (e. g. in the refrigerator). You are NOT supposed to pull at the attached wire in order to remove the electrodes neither from the protection foil nor from skin. This may cause damage of the electrodes. Some water should be dropped on the sticky side of the electrodes in case of re-using them. The electrodes should be thrown away if they do not adhere any longer. The electrodes should not be exposed to pressure.

Caution

Usage should be stopped immediately in case of skin irritation. Please ask your doctor for medical advice.